


# **ULTRA - PAP**

**Ultra Fast Papanicolaou Staining Kit**

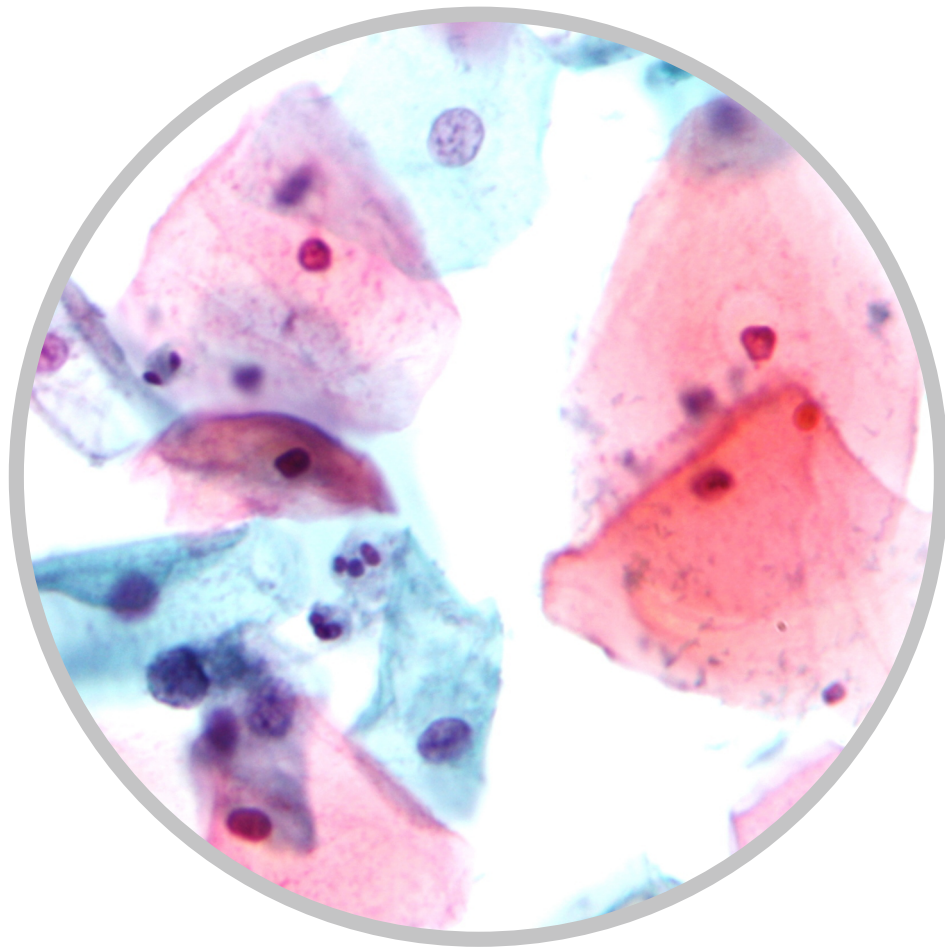

**Single source, standardized pap staining reagents !**

# ULTRA - PAP

## Ultra Fast Papanicolaou Staining Kit

### Presentation



Feature	Benefit
Complete and composite kit	All staining reagents available from single standardized source
Less than 3 minutes time for staining	Less time consuming
Results comparable with traditional papanicolaou stain	Confidence in reporting of results
Few steps in staining procedure	User friendly

### Staining Procedure :

- Hydrate the fixed smear with 10 passes under running tap water. Blot off excess water.
- Dip in Nuclear Stain for 45 seconds. Wash under running tap water.
- Add 3 drops of Scotts Tap Water Buffer.
- Wash in running tap water after 10 seconds until dye traces are removed. Blot off excess water.
- Dip in Dehydrant 30 seconds. (Two changes)
- Dip in Working Cyto-Stain 15 seconds. Wash in running tap water. Blot off excess water.
- Dip in Dehydrant for 30 seconds. Remove and allow to dry.
- Dip in Xylene (Two changes if required) and allow to dry.
- Add DPX Mountant and cover with a cover slip and observe under microscope.

**Note:** In case Coplin jars are not used one can flood the slide with a few drops of stains as required.

**Pack Size :** Reagents sufficient for preparing 250 smears and 600 smears.



**Microxpress®**

An ISO 13485  
Certified Company

A Division of

**T TULIP DIAGNOSTICS (P) LTD.**

Plot No. S-124, S-125, S-126, Utility Plot No. VIII, Phase III-B, Verna Industrial Estate, Verna, Goa - 403 722.

**Regd. Office:** Gitanjali, Tulip Block, Dr. Antonio do Rego Bagh, Alto Santacruz,  
Bambolim Complex Post Office, Goa - 403 202, INDIA.

**Email:** mex.queries@tulipgroup.com; **Website:** www.microxpress.in