


ULTRA - PAP

Ultra Fast Papanicolaou Staining Kit

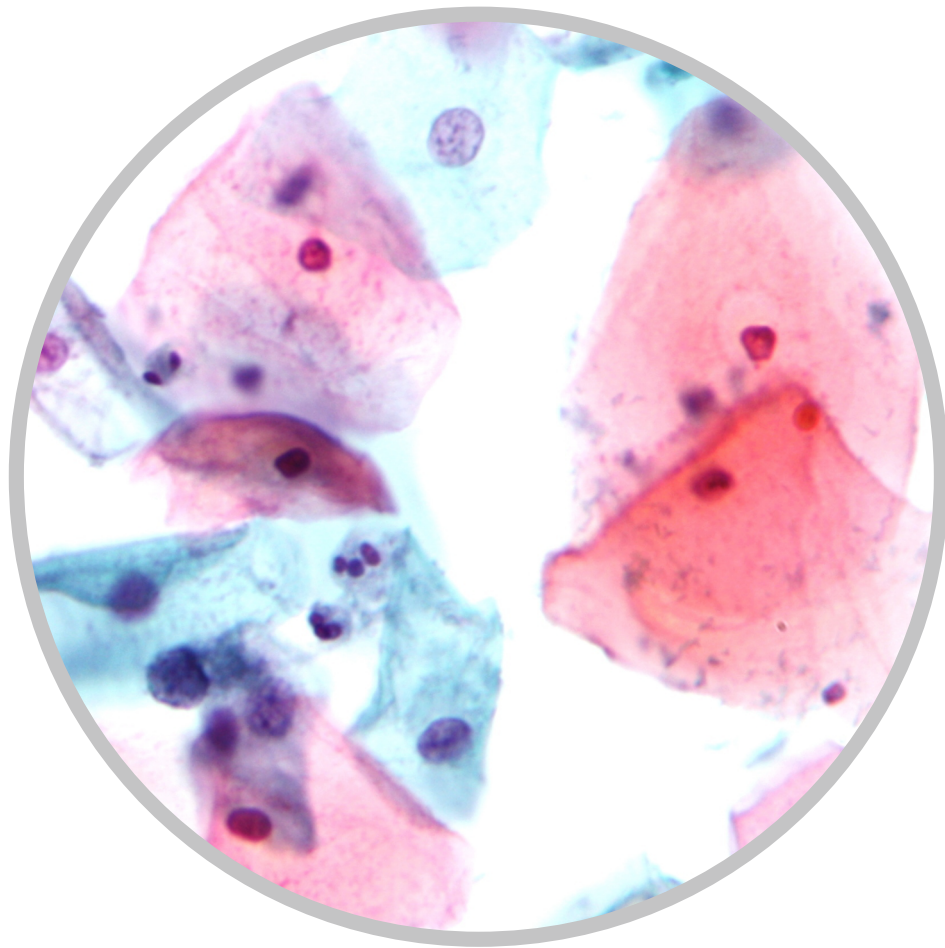

Single source, standardized pap staining reagents !

ULTRA - PAP

Ultra Fast Papanicolaou Staining Kit

Presentation



Feature	Benefit
Complete and composite kit	All staining reagents available from single standardized source
Less than 3 minutes time for staining	Less time consuming
Results comparable with traditional papanicolaou stain	Confidence in reporting of results
Few steps in staining procedure	User friendly

Staining Procedure :

- Hydrate the fixed smear with 10 passes under running tap water. Blot off excess water.
- Dip in Nuclear Stain for 45 seconds. Wash under running tap water.
- Add 3 drops of Scotts Tap Water Buffer.
- Wash in running tap water after 10 seconds until dye traces are removed. Blot off excess water.
- Dip in Dehydrant 30 seconds. (Two changes)
- Dip in Working Cyto-Stain 15 seconds. Wash in running tap water. Blot off excess water.
- Dip in Dehydrant for 30 seconds. Remove and allow to dry.
- Dip in Xylene (Two changes if required) and allow to dry.
- Add DPX Mountant and cover with a cover slip and observe under microscope.

Note: In case Coplin jars are not used one can flood the slide with a few drops of stains as required.

Pack Size : Reagents sufficient for preparing 250 smears and 600 smears.

An ISO 13485 Certified Company
Micropress[®]

A Division of

T TULIP DIAGNOSTICS (P) LTD.

Plot No. S-124, S-125, S-126, Utility Plot No. VIII, Phase III-B, Verna Industrial Estate, Verna, Goa - 403 722.

Regd. Office: Gitanjali, Tulip Block, Dr. Antonio do Rego Bagh, Alto Santacruz,
 Bambolim Complex Post Office, Goa - 403 202, INDIA.

Email: mex.queries@tulipgroup.com; **Website:** www.micropress.in